

RE VITA LIZE

MESOTHERAPY
PROTOCOLS



REALIZE YOUR INNER BEAUTY

HIGH CONCENTRATION OF ACTIVE INGREDIENTS
INNOVATIVE | UNIQUE



GLOW INSIDE-OUT AND OWN YOUR CONFIDENCE ALONG THE BEAUTY JOURNEY!

The Beauty Journey encourages you and your clients to combine great skin care, effective rejuvenation, and superb facial contouring to enhance their natural beauty. By promoting the Beauty Journey circle in your clinic, you are offering clients the full package.

WHAT IS THE SECRET OF THE REVITALIZE SUCCESS?

- The 8 different serum cocktails are compatible with most cosmetic and aesthetic protocols and equipment
- Immediate spectacular results can be achieved through their natural and balanced cocktails ingredients
- Various skin irregularities can be solved (scars, acne and dark spots)
- Unique innovation at your fingertips to enhance treatment results
- Most of the serums can be mixed to tailor treatments to your client's individual requirements
- Color codes make choosing the right serums easy
- Lovable design with powerful messages
- Our success comes from YOU - Innovation based on feedback from **Revitalize** experts



MY MED AESTHETICS IS DEDICATED TO ELEVATING YOU AND YOUR BUSINESS!

Medical aesthetics professionals, doctors and beauticians around the world offer you the most innovative products with the mission to see more beauty around us. We are devoted to reaching out to the best beauty artists that are feeling the call to grow together while helping people in slow aging with humility and harmony.

COCKTAILS. A MIX FOR EVERY SCENARIO

The award-winning Revitalize serums on their own give exceptional results, but in order to maximize the outcome according to your patient's skin and needs, you can create serum cocktails. It's quick and easy to do and gives you the opportunity to offer your patients truly magical results.

Combination	Vials/ampules used	Indication
Hair Rejuvenation cocktail	1 amp. Booster : 1 vial Hair	Diffuse hair loss, hair thinning, baldness
Scar Remover cocktail	2 amp. Booster : 1 vial Pure	Scars (acne scars, surgical scars, scars after injury) and active acne
Super-Vitalizer cocktail	1 vial Pure : 1 vial Vitamins	For tired, dull, lifeless, grey, skin that has lost its glow
Slimmer cocktail	1 vial Slim : 1 vial Body	Fat reduction and toning, tightening at the same time (arms, abdomen, buttocks double chin)
Tightening cocktail	1 vial Pure : 1 vial Body	Hydration and toning for a face that is slightly wrinkled and has lost its tone in the process
Illuminating cocktail	1 vial Pure : 1 vial Bright Skin	It hydrates and lightens at the same time in cases where hyperpigmentation has occurred
Vita-Light cocktail	1 vial Vitamins : 1 vial Bright Skin	Removes skin spots and nourishes the skin at the same time
Body-Vitalizer cocktail	1 vial Vitamins : 1 vial Body	Tightens skin with the addition of a nourishing effect





BOOSTER.

UNIQUE REGENERATION

Growth factors are naturally produced when a wound in our body requires tissue reparation. Fractional lasers, microneedling and electroporation are based on this exact same process, the practitioner creates a microscopic wound that triggers natural Epidermal Growth Factor production.



MYF506

REVITALIZE BOOSTER

Indication: It promotes dermal repair increasing its self-regeneration rate.

BOOSTER IS NOT INJECTABLE. ALWAYS MIX WITH PURE OR HAIR.

Contents:


4 ampoules with epidermal growth factor (EGF), an innovative protein that represents a revolution in beauty and regeneration treatments.

Packaging: 4 ampoules, 1 ml each.

PROTOCOL OF TREATMENT:

PREPARATION

- A patch test must to be carried out 24 hours before treatment (with a sterile needle pierce the skin superficially behind the ear or on the forearm apply a tiny droplet of the serum and leave in place for minimum 24 hours). **If the skin reacts do not proceed with the treatment.**
 - Clean and sterilise the chosen area to be treated.
- Mix the content of 2 ampoules of **Revitalize Booster** into a vial of **Revitalize Pure**.
 - Shake until an homogeneous emulsion forms.

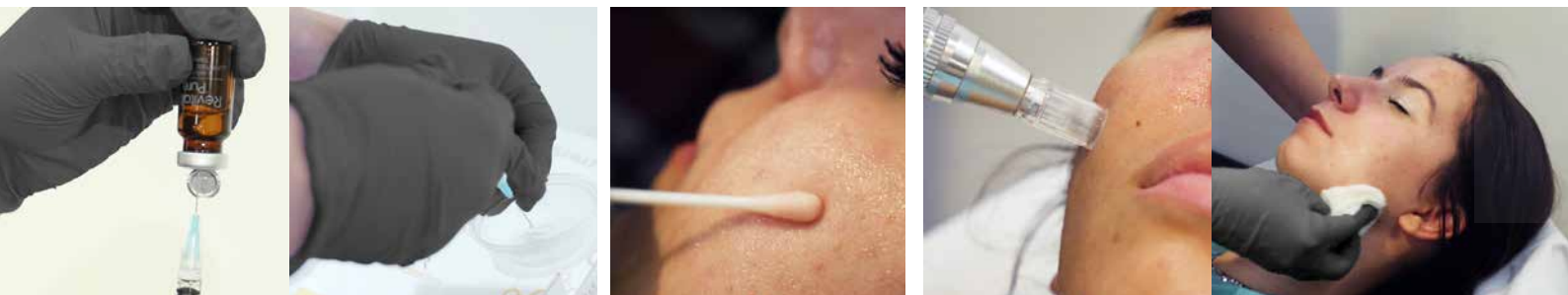


ACTIVE ACNE


One session per week for a maximum of 6 weeks.

TREATMENT


- 1** Apply with a cotton bud **0.5ml** of the solution at a **10cm x 10cm** area to be treated.




Amount of solution: **0.50ml**

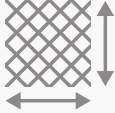


Treatment area: **10cm x 10cm**
- 2** Choose the depth on **Revitalize Pen** to **0.50mm** (in case of skin sensivity use only **0.25mm**) and start work with it in straight lines horizontally and vertically at the area to be treated.




Depth: **0.25-0.50mm**




Speed: **slow**




- 3** Apply another **0.5ml** of the solution at the same area.

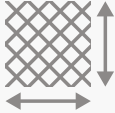

Amount of solution: **0.50ml**



Treatment area: **10cm x 10cm**
- 4** Work with the **Revitalize Pen** in **circle** or **"infinity" shapes** covering all the area with the **same depth**.

Revitalize Pen depth: **0.25-0.50mm**


Complete the treatment with an acne cleanser, a serum regulating vial and a home treatment cream.

ACNE SCARS


One session per week for minimum 6 weeks.

TREATMENT


- 1** Apply with a cotton bud **0.5ml** of the solution at a **10cm x 10cm** area to be treated.




Amount of solution: **0.50ml**

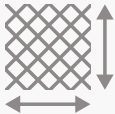


Treatment area: **10cm x 10cm**
- 2** Choose the depth on **Revitalize Pen** to **1-1.5mm** and start work with it in straight lines horizontally and vertically at the area to be treated.




Depth: **1-1.5mm**




Speed: **fast**




- 3** Apply another **0.5ml** of the solution at the same area.

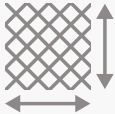

Amount of solution: **0.50ml**



Treatment area: **10cm x 10cm**
- 4** Work with the **Revitalize Pen** in **circle** or **"infinity" shapes** covering all the area with the **same depth**.

Revitalize Pen depth: **1-1.5mm**

BOOSTER.


UNIQUE REGENERATION

ANTI AGE


One session per week for a maximum of 6 weeks.

TREATMENT


- 1 Apply with a cotton bud **0.5ml** of the solution at a **10cm x 10cm** area to be treated.




Amount of solution: **0.50ml**

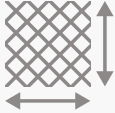


Treatment area: **10cm x 10cm**
- 2 Choose the depth on **Revitalize Pen** to **0.75mm** and start work with it in straight lines horizontally and vertically at the area to be treated.




Depth: **0.75mm**




Speed: **fast**



- 3 Apply another **0.5ml** of the solution at the same area previously chosen.

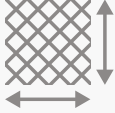


Amount of solution: **0.50ml**



Treatment area: **10cm x 10cm**
- 4 Work with the **Revitalize Pen** in **circle** or **"infinity" shapes** covering all the area with the **same depth**.



Revitalize Pen depth: **0.75mm**




REGENERATOR


- First 3 weeks: Revitalize Booster for one session per week.
- Second 3 weeks: Revitalize Vitamins for one session per week.

TREATMENT


- 1 Apply with a cotton bud **0.5ml** of the solution at a **10cm x 10cm** area to be treated.




Amount of solution: **0.50ml**

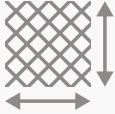


Treatment area: **10cm x 10cm**
- 2 Choose the depth on **Revitalize Pen** to **0.75mm** and start work with it in straight lines horizontally and vertically at the area to be treated.




Depth: **0.75mm**




Speed: **fast**



- 3 Apply another **0.5ml** of the solution at the same area.

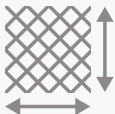


Amount of solution: **0.50ml**



Treatment area: **10cm x 10cm**
- 4 Work with the **Revitalize Pen** in **circle** or **"infinity" shapes** covering all the area with the **same depth**.



Revitalize Pen depth: **0.75mm**



POST TREATMENT RECOMMENDATIONS

1. Do not touch the treated area.
2. Wear loose clothing over any treated area.
3. No makeup, skin preparations or abrasive products for 24 hours.
4. Avoid skin/hair preparations for 24 hours.
5. Expect swelling redness and bruising (these are temporary).
6. If you experience undue discomfort contact your practitioner immediately.
7. Expect some mild irritation and bumps under the skin (this is normal and will disperse).
8. Keep the treated area dry for 24 hours.
9. Avoid alcohol, exercise and extreme temperature for 48 hours.
10. Stay out of the sun (cover the area if going outside).
11. Avoid sun-beds, saunas and steam rooms.
12. Contact your practitioner if you have any concerns following treatment.

Following **Revitalize Booster** clinical trials microneedling techniques obtain the best aesthetic results with less trauma for the patient.

Revitalize Booster can be used with other mesotherapy techniques such as High Frequency (HiFu), Electroporation and Mesogun. Please note that other techniques may require a different setting. Refer to your training manual.

Precautions:

Topical use only, **DO NOT INJECT**. Avoid contact with eyes. In case of contact with eyes, rinse immediately with plenty of water. It is not recommended to use this product if you are allergic to any of its ingredients. Keep out of children's reach. Store under 25°C in a dry place

Do not expose to direct sunlight.





ESSENCE.

IMMEDIATE AND
LONG-LASTING RESULTS

Revitalize Essence has an optimized potential to hydrate the tissues with added cross-link technology to give much longer-lasting results. It has both high and low-density molecules perfectly balanced to give optimum hydration coupled with a robust viscosity. This combination makes it a 'hybrid' product offering clients the rejuvenating properties of a top-quality mesotherapy serum whilst physically smoothing wrinkles and skin imperfections in a similar way that dermal fillers work within the tissues.

REVITALZE ESSENCE

Indication: Restores elasticity and promotes longer-lasting hydration.

Application:

Apply **Revitalize Essence** to the hands, face, neck, décolletage and body.

1st Month: 2 applications (once every 2 weeks).

2nd and 3rd Month: 1 application (once a month).

Maintenance: 1 application every 60 days.

Contents:

- **Sodium Hyaluronate Crosspolymer** - 52,5 mg/vial
- **Sodium Acetylated Hyaluronate** - 6,5 mg/vial
- **Hydrolyzed Sodium Hyaluronate** - 1 mg/vial
- **Sodium Hyaluronate** - 5 mg/vial



MYF515

Packaging: 2 vials, 5 ml each.



A patch test must to be carried out 24 hours before treatment (with a sterile needle pierce the skin superficially behind the ear or on the forearm apply a tiny droplet of the serum and leave in place for minimum 24 hours). **If the skin reacts do not proceed with the treatment.**

REVITALIZE PROTOCOL

0

Cleanse the skin thoroughly and apply anaesthetic topical cream to the treatment area.

After **15-20 minutes** check if the area is numb. Remove topical cream with gauze and warm water. Double cleanse using a suitable skin cleansing solution such as a micellar cleanser & blot dry. Mask the treatment area using clean gauze.

1

Draw up **1ml** of **Revitalize Essence** per **10x10 cm** area using a sterile syringe and needle and then remove the needle (two serums may be mixed together forming a cocktail).



Amount of solution:
1ml per 10 x 10 cm

2

Deposit droplets of the serum onto the treatment area & spread evenly using a cotton bud or a sterile gloved finger. **0.5ml** at the first application.



3

Choose depth on **Revitalize Pen** to maximum depth (**0.25-1.5mm on the face** or **2-2.5mm on the body**) using the maximum speed. Begin to stamp the serum into the skin covering all areas using vertical & horizontal lines.



Depth:
Face: **0.25-1.5mm**
Body: **2-2.5mm**
Speed: **fast**

4

Reduce the depth and the speed of the **Revitalize Pen** to **50%** previously chosen. Deposit the remaining **0.5ml** serum onto the skin. Use the **Revitalize Pen** in a circular motion across the treatment area.



Depth: **reduce by 50%**
Speed: **medium**

5

Gently massage any remaining serum into the skin. Allow to dry naturally. Use a high SPF factor cream following treatment to protect from UV light.

Post treatment advice: Keep the treated area dry for 24 hours & always wear a high SPF cream.





PURE.

THE EXPERT IN SKIN
IMPERFECTIONS

Stretch marks, acne marks, scar marks and other skin imperfections are undesirable for anyone from an aesthetic point of view. The Revitalize Pure unique formulation has the perfect mixture and balance of collagen with hyaluronic acid. It increases the generation of new collagen fibres, repairs and reinforces the existing ones.

A repairing and restructuring treatment which was dreamed about, is now a reality...

REVITALZE PURE

Indication: For all skins that need to regenerate the texture and tonality.

Application:

Apply **Revitalize Pure** to the hands, face, neck and neckline, with electroporation techniques or radiofrequency. The microneedling technique can considerably speed up the results.

Month 1: 1 session every 15/30 days.
Month 2-3: 1 session every 30/60 days according to evolution.
Maintenance: 1 session every 60 days.

Contents:

- **Sodium Hyaluronate** - 12mg/ml
- **Hydrolyzed Collagen** - 5%

Packaging: 5 vials, 5 ml each.



MYF500



A patch test must to be carried out 24 hours before treatment (with a sterile needle pierce the skin superficially behind the ear or on the forearm apply a tiny droplet of the serum and leave in place for minimum 24 hours). **If the skin reacts do not proceed with the treatment.**

REVITALIZE PROTOCOL

0

Cleanse the skin thoroughly and apply anaesthetic topical cream to the treatment area.

After **15-20 minutes** check if the area is numb. Remove topical cream with gauze and warm water. Double cleanse using a suitable skin cleansing solution such as a micellar cleanser & blot dry. Mask the treatment area using clean gauze.

1

Draw up **1ml** of **Revitalize Pure** per **10x10 cm** area using a sterile syringe and needle and than remove the needle (two serums may be mixed together forming a cocktail).



Amount of solution:
1ml per 10 x 10 cm

2

Deposit droplets of the serum onto the treatment area & spread evenly using a cotton bud or a sterile gloved finger. **0.5ml** at the first application.



3


Choose depth on **Revitalize Pen** to maximum depth (**0.25-1.5mm on the face** or **2-2.5mm on the body**) using the maximum speed. Begin to stamp the serum into the skin covering all areas using vertical & horizontal lines.



Depth:
Face: **0.25-1.5mm**
Body: **2-2.5mm**
Speed: **fast**

4

Reduce the depth and the speed of the **Revitalize Pen** to **50%** previously chosen. Deposit the remaining **0.5ml** serum onto the skin. Use the **Revitalize Pen** in a circular motion across the treatment area.

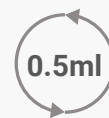

Depth: **reduce by 50%**
Speed: **medium**

5

Gently massage any remaining serum into the skin. Allow to dry naturally. Use a high SPF factor cream following treatment to protect from UV light.

Post treatment advice: Keep the treated area dry for 24 hours & always wear a high SPF cream.





VITAMINS.

NOURISH THE SKIN
FROM WITHIN

Stress, sun over-exposure and poor diet age the skin. To recover healthy skin, it must be treated correctly.

Revitalize Vitamins has a special formula composed of 69 essential nutrients such as Hyaluronic Acid, Retinol, Glutathione, disodium flavin adenine dinucleotide (FAD), biotin, Vitamin C, aminobutyric acid and taurine. Bring back the skin's beauty and maintain it.

REVITALIZE VITAMINS

Indication: Revitalizing and illuminating.

Application: Apply to the hands, face, neck and décolletage.

Month 1: 1 session every 7 days.
Month 2: 1 session every 15 days.
Maintenance: 1 session every 30 days.

Contents:

- Glutathione
- Niacinamide
- Vitamin C
- Proline
- Biotin
- Folic Acid
- Riboflavin
- Glycine
- HA - 25 mg/vial
- Leucine
- Glutamine
- Aspartic Acid
- Arginine HCL

Packaging: 5 vials, 5 ml each.



MYF500



A patch test must to be carried out 24 hours before treatment (with a sterile needle pierce the skin superficially behind the ear or on the forearm apply a tiny droplet of the serum and leave in place for minimum 24 hours). **If the skin reacts do not proceed with the treatment.**

REVITALIZE PROTOCOL

0

Cleanse the skin thoroughly and apply anaesthetic topical cream to the treatment area.

After **15-20 minutes** check if the area is numb. Remove topical cream with gauze and warm water. Double cleanse using a suitable skin cleansing solution such as a micellar cleanser & blot dry. Mask the treatment area using clean gauze.

1

Draw up **1ml** of **Revitalize Vitamins** per **10x10 cm** area using a sterile syringe and needle and than remove the needle (two serums may be mixed together forming a cocktail).



Amount of solution:
1ml per 10 x 10 cm

2

Deposit droplets of the serum onto the treatment area & spread evenly using a cotton bud or a sterile gloved finger. **0.5ml** at the first application.



3


Choose depth on **Revitalize Pen** to maximum depth (**0.25-1.5mm on the face** or **2-2.5mm on the body**) using the maximum speed. Begin to stamp the serum into the skin covering all areas using vertical & horizontal lines.



Depth:
Face: **0.25-1.5mm**
Body: **2-2.5mm**
Speed: **fast**

4

Reduce the depth and the speed of the **Revitalize Pen** to **50%** previously chosen. Deposit the remaining **0.5ml** serum onto the skin. Use the **Revitalize Pen** in a circular motion across the treatment area.

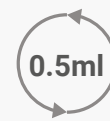

Depth: **reduce by 50%**
Speed: **medium**

5

Gently massage any remaining serum into the skin. Allow to dry naturally. Use a high SPF factor cream following treatment to protect from UV light.

Post treatment advice: Keep the treated area dry for 24 hours & always wear a high SPF cream.





BODY.

TIGHTENS SAGGING SKIN

Body and facial skin flaccidity is a direct consequence of aging and/or weight change. To maintain firm skin, without wrinkles, we must be consistent with a healthy diet and good hydration habits. Revitalize Body's formula is composed of organic compounds, DMAE and Dexpanthenol, proven to reaffirm and restore the collagen fibers that support the skin, re-shaping them and removing the sagging effect to combat flaccidity.

REVITALIZE BODY

Indications: Flaccidity, striae, stretch marks, post laser and post peeling.



MYF502

Application:

Apply to the hands, face, neck, décolletage and body.

Month 1: 1 session every 7 days.
Month 2: 1 session every 15 days.
Maintenance: 1 session every 30 days.

Contents:

- **Dimethylaminoethanol Tartrate** - 0,5%
- **Dexpanthenol** - 1%
- **Methylsilanol Mannuronate** - 0,05%

Packaging: 5 vials, 5 ml each.



A patch test must to be carried out 24 hours before treatment (with a sterile needle pierce the skin superficially behind the ear or on the forearm apply a tiny droplet of the serum and leave in place for minimum 24 hours). **If the skin reacts do not proceed with the treatment.**

REVITALIZE PROTOCOL

0

Cleanse the skin thoroughly and apply anaesthetic topical cream to the treatment area.

After **15-20 minutes** check if the area is numb. Remove topical cream with gauze and warm water. Double cleanse using a suitable skin cleansing solution such as a micellar cleanser & blot dry. Mask the treatment area using clean gauze.

1

Draw up **1ml of Revitalize Body** per **10x10 cm** area using a sterile syringe and needle and than remove the needle (two serums may be mixed together forming a cocktail).



Amount of solution:
1ml per 10 x 10 cm

2

Deposit droplets of the serum onto the treatment area & spread evenly using a cotton bud or a sterile gloved finger. **0.5ml** at the first application.



3


Choose depth on **Revitalize Pen** to maximum depth (**0.25-1.5mm on the face** or **2-2.5mm on the body**) using the maximum speed. Begin to stamp the serum into the skin covering all areas using vertical & horizontal lines.



Depth:
Face: **0.25-1.5mm**
Body: **2-2.5mm**
Speed: **fast**

4

Reduce the depth and the speed of the **Revitalize Pen** to **50%** previously chosen. Deposit the remaining **0.5ml** serum onto the skin. Use the **Revitalize Pen** in a circular motion across the treatment area.

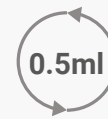

Depth: **reduce by 50%**
Speed: **medium**

5

Gently massage any remaining serum into the skin. Allow to dry naturally. Use a high SPF factor cream following treatment to protect from UV light.

Post treatment advice: Keep the treated area dry for 24 hours & always wear a high SPF cream.





HAIR.

STRENGTHENING HAIR MATRIX

Hair loss at an early age creates discomfort and embarrassment for anyone, man or woman.

Revitalize Hair acts in the prevention, re-densification, and stimulation of the hair folliculos matrix responsible for the hair, beard and eyebrow growth.

Moreover, **Revitalize Hair** acts at the dermal level on the fibroblasts situated next to the folliculos bulbs optimizing the development of the hair matrix.



MYF504

REVITALIZE HAIR

Indication: Nourishing the scalp and promoting hair growth.

Contents:

- Biotin
- Caffeine
- Panthenol
- Ginko Biloba
- Hyaluronic Acid
- Propylene Glycol
- Glutamine
- Glicine
- Arginine HCl
- Hydroxypropyl methylcellulose
- DHQC
- EGCG

Packaging:
5 vials, 10 ml each

PROTOCOL OF TREATMENT:

PREPARATION

- A patch test must to be carried out 24 hours before treatment (with a sterile needle pierce the skin superficially behind the ear or on the forearm apply a tiny droplet of the serum and leave in place for minimum 24 hours). **If the skin reacts do not proceed with the treatment.**
 - Clean and sterilise the chosen area to be treated.
- Mix the content of 2 ampoules of **Revitalize Booster** into a vial of **Revitalize Hair**.
 - Shake until an homogeneous emulsion forms.




HAIR REINFORCEMENT


One session per week for a maximum of 6 weeks.

TREATMENT


- 1** Remove the needle and use the syringe as a pipette to deposit droplets of the solution onto the treatment area. Spread the solution using a cotton bud **0.5ml** of the solution per **10 by 10cm** area.




Amount of solution: **0.5ml**

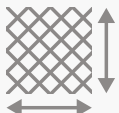


Treatment area: **10cm x 10cm**
- 2** Choose the depth on the revitalize pen up to **1mm** with the **highest speed** (in case of skin sensitivity use only **0.75mm**). Start working in straight lines horizontally and vertically.




Depth: **0.75-1mm**




Speed: **fast**




- 3** Apply another **0.5ml** of the solution at the same area.

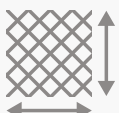

Amount of solution: **0.5ml**



Treatment area: **10cm x 10cm**
- 4** Work with the **Revitalize Pen** in a circular motion covering all the area with the **same depth and speed**.


Revitalize Pen depth: **0.75-1mm**

HAIR TRANSPLANT SUPPORT


One session per week for minimum 6 weeks.

TREATMENT


- 1** Remove the needle and use the syringe as a pipette to deposit droplets of the solution onto the treatment area. Spread the solution using a cotton bud **0.5ml** of the solution per **10 by 10cm** area.




Amount of solution: **0.5ml**

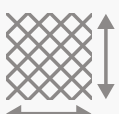


Treatment area: **10cm x 10cm**
- 2** Choose the depth on **Revitalize Pen** to **1mm** with **medium speed**. Start working in straight lines horizontally and vertically.




Depth: **1mm**




Speed: **medium**




- 3** Apply another **0.5ml** of the solution at the same area.

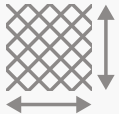

Amount of solution: **0.5ml**



Treatment area: **10cm x 10cm**
- 4** Work with the **Revitalize Pen** in a circular motion covering all the area with the **same depth and speed**.

Revitalize Pen depth: **1mm**

HAIR.


STRENGTHENING HAIR MATRIX

BEARD


One session per week for a maximum of 6 weeks.

TREATMENT


- Remove the needle and use the syringe as a pipette to deposit droplets of the solution onto the treatment area. Spread the solution using a cotton bud **0.5ml** of the solution per **10 by 10cm** area.




Amount of solution: **0.5ml**

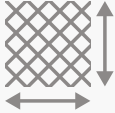


Treatment area: **10cm x 10cm**
- Choose the depth on **Revitalize Pen** to **0.75mm** with the **highest speed**. Start working in straight lines horizontally and vertically.




Depth: **0.75mm**




Speed: **fast**




- Apply another **0.5ml** of the solution at the same area.

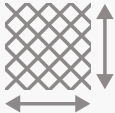

Amount of solution: **0.5ml**



Treatment area: **10cm x 10cm**
- Work with the **Revitalize Pen** in a circular motion covering all the area with the **same depth** and **speed**.




Revitalize Pen depth: **0.75mm**

EYEBROW

- First 3 weeks: Revitalize Booster and Hair for one session per week.
- Second 3 weeks: Revitalize Hair for one session per week.

TREATMENT


- Remove the needle and use the syringe as a pipette to deposit droplets of the solution onto the treatment area. Spread the solution using a cotton bud **0.5ml** of the solution per **10 by 10cm** area.




Amount of solution: **0.5ml**




Treatment area: **10cm x 10cm**
- Choose the depth on **Revitalize Pen** to **0.5mm** with the **lowest speed**. Start working in straight lines horizontally.



Depth: **0.5mm**




Speed: **slow**



- Apply another **0.5ml** of the solution at the same area.

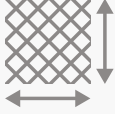

Amount of solution: **0.5ml**



Treatment area: **10cm x 10cm**
- Work with the **Revitalize Pen** in straight lines horizontally covering the same area with the **same depth** and **speed**.



Revitalize Pen depth: **0.5mm**

POST TREATMENT RECOMMENDATIONS

1. Do not touch the treated area.
2. Avoid skin/hair preparations for 24 hours.
3. Expect swelling redness and bruising (these are temporary).
4. If you experience undue discomfort contact your practitioner immediately.
5. Expect some mild irritation and bumps under the skin (this is normal and will disperse).
6. Keep the treated area dry for 24 hours.
7. Avoid alcohol, exercise and extreme temperature for 48 hours.
8. Stay out of the sun (cover the area if going outside).
9. Avoid sun-beds, saunas and steam rooms.
10. Gently wash with a mild shampoo after 24 hours.
11. Do not use abrasive combs or hairbrushes.
12. Contact your practitioner if you have any concerns following treatment.

Following **Revitalize Hair** clinical trials microneedling techniques obtain the best aesthetic results with less trauma for the patient.

Revitalize Hair can be used with other mesotherapy techniques such as High Frequency (HiFu), Electroporation and Mesogun. Please note that other techniques may require a different setting. Refer to your training manual.

Precautions:

Topical use only, **DO NOT INJECT**. Avoid contact with eyes. In case of contact with eyes, rinse immediately with plenty of water. It is not recommended to use this product if you are allergic to any of its ingredients. Keep out of children's reach. Store under 25°C in a dry place

Do not expose to direct sunlight.





SLIM.

TARGETED FAT
REDUCTION

Poor diet, lack of exercise and stress are the main reasons for fat accumulation. It has become one of the main requests in aesthetic clinics. Revitalize Slim's lipolytic and anti-cellulite formula comprises an insulin-like growth factor, phosphatidylcholine, sodium deoxycholate, carnitine and trace elements.

REVITALIZE SLIM

Indications: Localized fat accumulation spot as tummy, buttocks, back of the arm, thighs and double chin.

Application:

A weekly or a fortnightly application of **Revitalize Slim** is recommended through therapy with microneedle and/or virtual mesotherapy for electroporation.

Month 1: 1 session every 15/30 days.

Month 2: 1 session every 30/60 days according to evolution.

Contents:

- **Phosphatidylcholine** - 3%
- **Sodium Deoxycholate** - 2%
- **Carnitine**
- **Insulin like growth factor**

Packaging: 5 vials, 10 ml each.



MYF503



A patch test must to be carried out 24 hours before treatment (with a sterile needle pierce the skin superficially behind the ear or on the forearm apply a tiny droplet of the serum and leave in place for minimum 24 hours). **If the skin reacts do not proceed with the treatment.**

REVITALIZE PROTOCOL

0

Cleanse the skin thoroughly and apply anaesthetic topical cream to the treatment area.

After **15-20 minutes** check if the area is numb. Remove topical cream with gauze and warm water. Double cleanse using a suitable skin cleansing solution such as a micellar cleanser & blot dry. Mask the treatment area using clean gauze.

1

Draw up **1ml** of **Revitalize Slim** per **10x10 cm** area using a sterile syringe and needle and than remove the needle (two serums may be mixed together forming a cocktail).



Amount of solution:
1ml per 10 x 10 cm

2

Deposit droplets of the serum onto the treatment area & spread evenly using a cotton bud or a sterile gloved finger. **0.5ml** at the first application.



3


Choose depth on **Revitalize Pen** to maximum depth (**0.25-1.5mm on the face** or **2-2.5mm on the body**) using the maximum speed. Begin to stamp the serum into the skin covering all areas using vertical & horizontal lines.



Depth:
Face: **0.25-1.5mm**
Body: **2-2.5mm**
Speed: **fast**

4

Deposit the remaining **0.5ml** serum onto the skin. Use the **Revitalize Pen** in a circular motion across the treatment area.

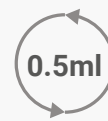

Depth:
Face: **0.25-1.5mm**
Body: **2-2.5mm**
Speed: **fast**

5

Gently massage any remaining serum into the skin. Allow to dry naturally. Use a high SPF factor cream following treatment to protect from UV light.

Post treatment advice: Keep the treated area dry for 24 hours & always wear a high SPF cream.





BRIGHT SKIN.

LIGHTENING HYPERPIGMENTATION

Superficial pigmentation can be easily removed but deeper pigmentation requires a particular procedure. Mesotherapy treatments allow practitioners to reach the lower layers of the skin where pigmentation lies. Using Revitalize Bright Skin serum in combination with specialized devices will increase the freshness and glow of the skin.

REVITALIZE BRIGHT SKIN

Indications: Lighten skin tone and reduce pigmentation spots.

Application: Apply to the hands, face, neck, décolletage and body.

Contra-indications: Do not use on Rosacea.

1st and 2nd Month: 8 applications (once a week).

If the final results achieved before the 8 applications, the protocol can be suspended. The process can speed up with special techniques (E.g.: Revitalize Booster).

Contents:

- Kojic Acid - 0,9%
- PEG-35 Castor Oil - 0,4%
- Glycyrrhiza Glabra
- Root Extract - 0,3%
- Aspergillus Ferment
- Retinol
- Salicylic Acid
- Hyaluronic acid
- Morus Alba
- Tocopherol

Packaging: 5 vials, 5 ml each.



MYF505



A patch test must to be carried out 24 hours before treatment (with a sterile needle pierce the skin superficially behind the ear or on the forearm apply a tiny droplet of the serum and leave in place for minimum 24 hours). **If the skin reacts do not proceed with the treatment.**

REVITALIZE PROTOCOL

0

Cleanse the skin thoroughly and apply anaesthetic topical cream to the treatment area.

After **15-20 minutes** check if the area is numb. Remove topical cream with gauze and warm water. Double cleanse using a suitable skin cleansing solution such as a micellar cleanser & blot dry. Mask the treatment area using clean gauze.

1

Draw up **1ml** of **Revitalize Bright Skin** per **10x10 cm** area using a sterile syringe and needle and then remove the needle (two serums may be mixed together forming a cocktail).



Amount of solution:
1ml per 10 x 10 cm

2

Deposit droplets of the serum onto the treatment area & spread evenly using a cotton bud or a sterile gloved finger. **0.5ml** at the first application.



3


Choose depth on **Revitalize Pen** to maximum depth (**0.25-1.5mm on the face** or **2-2.5mm on the body**) using the maximum speed. Begin to stamp the serum into the skin covering all areas using vertical & horizontal lines.



Depth:
Face: **0.25-1.5mm**
Body: **2-2.5mm**
Speed: **fast**

4

Reduce the depth and the speed of the **Revitalize Pen** to **50%** previously chosen. Deposit the remaining **0.5ml** serum onto the skin. Use the **Revitalize Pen** in a circular motion across the treatment area.

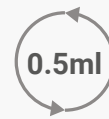

Depth: **reduce by 50%**
Speed: **medium**

5

Gently massage any remaining serum into the skin. Allow to dry naturally. Use a high SPF factor cream following treatment to protect from UV light.


Post treatment advice: Keep the treated area dry for 24 hours & always wear a high SPF cream.





 www.revitalize.eu


 info@my-med.eu

 +36 70 389 5841

 @mymedaesthetic

 mymedaesthetics

 MyMedVideos

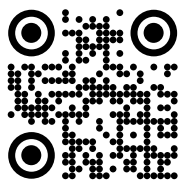

Téglagyári út 5-11.
2097 Pilisborosjenő, Hungary

www.my-med.eu

