



# BOOSTER.

UNIQUE REGENERATION

Growth factors are naturally produced when a wound in our body requires tissue reparation. Fractional lasers, microneedling and electroporation are based on this exact same process, the practitioner creates a microscopic wound that triggers natural Epidermal Growth Factor production.



MYF506

## REVITALIZE BOOSTER

**Indication:** It promotes dermal repair increasing its self-regeneration rate.

**BOOSTER IS NOT INJECTABLE. ALWAYS MIX WITH PURE OR HAIR.**

**Contents:**


4 ampoules with epidermal growth factor (EGF), an innovative protein that represents a revolution in beauty and regeneration treatments.

**Packaging:** 4 ampoules, 1 ml each.

## PROTOCOL OF TREATMENT:

### PREPARATION

- A patch test must to be carried out 24 hours before treatment (with a sterile needle pierce the skin superficially behind the ear or on the forearm apply a tiny droplet of the serum and leave in place for minimum 24 hours). **If the skin reacts do not proceed with the treatment.**
  - Clean and sterilise the chosen area to be treated.
- Mix the content of 2 ampoules of **Revitalize Booster** into a vial of **Revitalize Pure**.
  - Shake until an homogeneous emulsion forms.

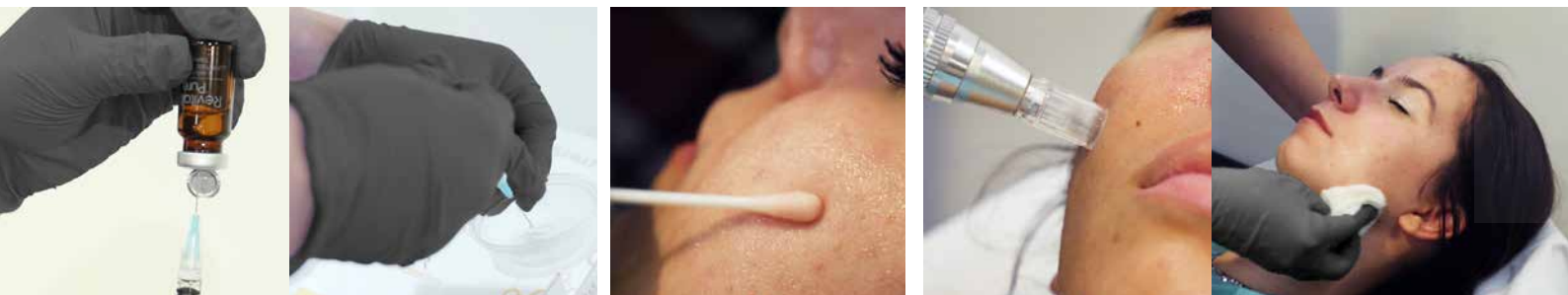


## ACTIVE ACNE


One session per week for a maximum of 6 weeks.

### TREATMENT


- 1** Apply with a cotton bud **0.5ml** of the solution at a **10cm x 10cm** area to be treated.




Amount of solution: **0.50ml**

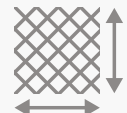


Treatment area: **10cm x 10cm**
- 2** Choose the depth on **Revitalize Pen** to **0.50mm** (in case of skin sensivity use only **0.25mm**) and start work with it in straight lines horizontally and vertically at the area to be treated.




Depth: **0.25-0.50mm**




Speed: **slow**




- 3** Apply another **0.5ml** of the solution at the same area.

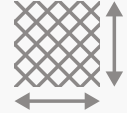

Amount of solution: **0.50ml**



Treatment area: **10cm x 10cm**
- 4** Work with the **Revitalize Pen** in **circle** or **"infinity" shapes** covering all the area with the **same depth**.

Revitalize Pen depth: **0.25-0.50mm**


Complete the treatment with an acne cleanser, a serum regulating vial and a home treatment cream.

## ACNE SCARS


One session per week for minimum 6 weeks.

### TREATMENT


- 1** Apply with a cotton bud **0.5ml** of the solution at a **10cm x 10cm** area to be treated.




Amount of solution: **0.50ml**

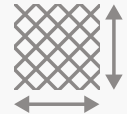


Treatment area: **10cm x 10cm**
- 2** Choose the depth on **Revitalize Pen** to **1-1.5mm** and start work with it in straight lines horizontally and vertically at the area to be treated.

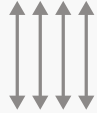


Depth: **1-1.5mm**




Speed: **fast**




- 3** Apply another **0.5ml** of the solution at the same area.

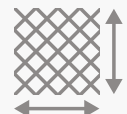

Amount of solution: **0.50ml**



Treatment area: **10cm x 10cm**
- 4** Work with the **Revitalize Pen** in **circle** or **"infinity" shapes** covering all the area with the **same depth**.

Revitalize Pen depth: **1-1.5mm**

# BOOSTER.


UNIQUE REGENERATION

## ANTI AGE


One session per week for a maximum of 6 weeks.

### TREATMENT


- Apply with a cotton bud **0.5ml** of the solution at a **10cm x 10cm** area to be treated.




Amount of solution: **0.50ml**

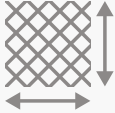


Treatment area: **10cm x 10cm**
- Choose the depth on **Revitalize Pen** to **0.75mm** and start work with it in straight lines horizontally and vertically at the area to be treated.




Depth: **0.75mm**




Speed: **fast**




- Apply another **0.5ml** of the solution at the same area previously chosen.

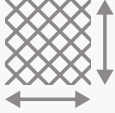

Amount of solution: **0.50ml**



Treatment area: **10cm x 10cm**
- Work with the **Revitalize Pen** in **circle** or **"infinity" shapes** covering all the area with the **same depth**.




Revitalize Pen depth: **0.75mm**

## REGENERATOR


- First 3 weeks: Revitalize Booster for one session per week.
- Second 3 weeks: Revitalize Vitamins for one session per week.

### TREATMENT


- Apply with a cotton bud **0.5ml** of the solution at a **10cm x 10cm** area to be treated.




Amount of solution: **0.50ml**

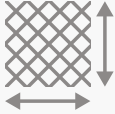


Treatment area: **10cm x 10cm**
- Choose the depth on **Revitalize Pen** to **0.75mm** and start work with it in straight lines horizontally and vertically at the area to be treated.




Depth: **0.75mm**




Speed: **fast**




- Apply another **0.5ml** of the solution at the same area.

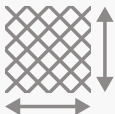

Amount of solution: **0.50ml**



Treatment area: **10cm x 10cm**
- Work with the **Revitalize Pen** in **circle** or **"infinity" shapes** covering all the area with the **same depth**.



Revitalize Pen depth: **0.75mm**

## POST TREATMENT RECOMMENDATIONS

1. Do not touch the treated area.
2. Wear loose clothing over any treated area.
3. No makeup, skin preparations or abrasive products for 24 hours.
4. Avoid skin/hair preparations for 24 hours.
5. Expect swelling redness and bruising (these are temporary).
6. If you experience undue discomfort contact your practitioner immediately.
7. Expect some mild irritation and bumps under the skin (this is normal and will disperse).
8. Keep the treated area dry for 24 hours.
9. Avoid alcohol, exercise and extreme temperature for 48 hours.
10. Stay out of the sun (cover the area if going outside).
11. Avoid sun-beds, saunas and steam rooms.
12. Contact your practitioner if you have any concerns following treatment.

Following **Revitalize Booster** clinical trials microneedling techniques obtain the best aesthetic results with less trauma for the patient.

**Revitalize Booster** can be used with other mesotherapy techniques such as High Frequency (HiFu), Electroporation and Mesogun. Please note that other techniques may require a different setting. Refer to your training manual.

### Precautions:

Topical use only, **DO NOT INJECT**. Avoid contact with eyes. In case of contact with eyes, rinse immediately with plenty of water. It is not recommended to use this product if you are allergic to any of its ingredients. Keep out of children's reach. Store under 25°C in a dry place

Do not expose to direct sunlight.

