



# VITAMINS.

NOURISH THE SKIN  
FROM WITHIN

Stress, sun over-exposure and poor diet age the skin. To recover healthy skin, it must be treated correctly.

Revitalize Vitamins has a special formula composed of 69 essential nutrients such as Hyaluronic Acid, Retinol, Glutathione, disodium flavin adenine dinucleotide (FAD), biotin, Vitamin C, aminobutyric acid and taurine. Bring back the skin's beauty and maintain it.

## REVITALIZE VITAMINS

**Indication:** Revitalizing and illuminating.

**Application:** Apply to the hands, face, neck and décolletage.

**Month 1:** 1 session every 7 days.  
**Month 2:** 1 session every 15 days.  
**Maintenance:** 1 session every 30 days.

**Contents:**

- Glutathione
- Niacinamide
- Vitamin C
- Proline
- Biotin
- Folic Acid
- Riboflavin
- Glycine
- HA - 25 mg/vial
- Leucine
- Glutamine
- Aspartic Acid
- Arginine HCL

**Packaging:** 5 vials, 5 ml each.



MYF500



A patch test must to be carried out 24 hours before treatment (with a sterile needle pierce the skin superficially behind the ear or on the forearm apply a tiny droplet of the serum and leave in place for minimum 24 hours). **If the skin reacts do not proceed with the treatment.**

## REVITALIZE PROTOCOL

0

Cleanse the skin thoroughly and apply anaesthetic topical cream to the treatment area.

After **15-20 minutes** check if the area is numb. Remove topical cream with gauze and warm water. Double cleanse using a suitable skin cleansing solution such as a micellar cleanser & blot dry. Mask the treatment area using clean gauze.

1

Draw up **1ml** of **Revitalize Vitamins** per **10x10 cm** area using a sterile syringe and needle and than remove the needle (two serums may be mixed together forming a cocktail).



Amount of solution:  
**1ml per 10 x 10 cm**

2

Deposit droplets of the serum onto the treatment area & spread evenly using a cotton bud or a sterile gloved finger. **0.5ml** at the first application.



3


Choose depth on **Revitalize Pen** to maximum depth (**0.25-1.5mm on the face** or **2-2.5mm on the body**) using the maximum speed. Begin to stamp the serum into the skin covering all areas using vertical & horizontal lines.



Depth:  
Face: **0.25-1.5mm**  
Body: **2-2.5mm**  
Speed: **fast**

4

Reduce the depth and the speed of the **Revitalize Pen** to **50%** previously chosen. Deposit the remaining **0.5ml** serum onto the skin. Use the **Revitalize Pen** in a circular motion across the treatment area.

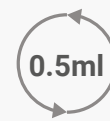

Depth: **reduce by 50%**  
Speed: **medium**

5

Gently massage any remaining serum into the skin. Allow to dry naturally. Use a high SPF factor cream following treatment to protect from UV light.

**Post treatment advice:** Keep the treated area dry for 24 hours & always wear a high SPF cream.

